

Ease Your Cashflow Use EasiFlo

**"I use EasiFlo when pumps are not available
and when precision is important"**

Tracy, Ward Manager



Simple to install

2 handed operation
for security

Reduces the risk of
over-infusing

Cost Effective in
comparison to using a
volumetric pump

**Why not use our 'EasiFlo' flow regulator for your low
risk infusions and reduce your consumables spend.**

Call the Sales Office on 01254 268510

EASIFLO I.V. FLOW REGULATOR

The EasiFlo is used for the delivery of low risk, non-viscous clear infusions.

It is simple to install and delivers your infusions with a graduated flow-rate of 5-250ml per hour with an accuracy of +/-10%. The Flow regulator is a two handed operation offering security and its silicone joint allows for continuous and precise flow control. The EF 300 is latex free

Precautions:

- The flow rate should be verified by drop counting
- The regulator has been calibrated for normal parenteral solutions. It should not be used with blood, blood products or glucose solutions above 10% or lipid emulsions.

Feedback from our customers:-

- More convenient for low risk infusions why order a pump from medical equipment library?
- More convenient for mobile patients than carrying a pump around on a drip stand
- Less expensive than pump consumables for hydration, delivery of antibiotics and potassium based infusions
- Great for subcutaneous infusions where patients, particularly the elderly have poor venous access.

FLOW RATE ml/hr	DROPS PER MIN. APPROX
5	2
10	3
15	5
20	7
25	8
30	10
40	13
50	15
60	20
80	25
100	30
125	40
150	50
200	66
250	80

